



MultiNeedling Pro

with dual functions


JEAN D'ARCEL
TECHNOLOGIE



Medical Beauty



Health
made in Germany




BEAUTY IS OUR PROFESSION

The Jean D'Arcel MultNeedling Pro™ is a revolutionary micro-needling device which aims to tighten, lift and rejuvenate skin.

Jean D'Arcel MultNeedling Pro™ can visibly improve the appearance of:



- **Fine lines**
- **Wrinkles**
- **Surgical, acne and trauma scars**
- **Stretch marks**
- **Pigmentation**
- **Pore size reduction**

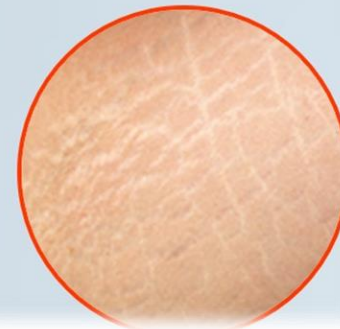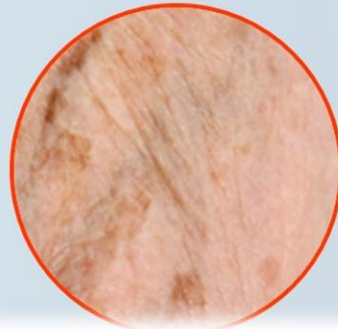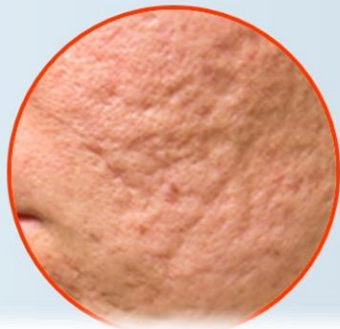

How does MultiNeedling Pro Works?

- Ultra-fine “MESO” tips punch micro-openings into the epidermis. Your natural protection function is maintained.
- The “MESO” tips move up and down fully automated, similar to a stamp. Thus, the epidermis is penetrated precisely and evenly. **Completely painless, gentle and fast.**
- Jean D'Arcel ampules active concentrated ingredients such as: ArcelMED Dermal Lift Repair, Hyaluron Booster, Hyaluron Filler, Mimic Lift Concentrate® , etc.... penetrate directly and evenly. In this way, they can develop their effect where it is important; in the deeper structures of the skin.
- This needling is offered with both plastic and stainless steel needles.

Application with the MultiNeedling Pro, is suitable for:

- The face
- Neck
- Décolleté
- Back of the hand
- The application at the eye area allows wrinkles to be visibly reduced.



How does MultiNeedlig Pro should be used?

Step 1:

Clean the face with a mild cleansing foam or gel. Use a non-refatting product for cleaning.

Step 2:

Peel the skin according to your needs and skin type with a gentle peeling product, for example with a light fruit acid or enzyme peeling which does not have to be vaporized. For intensive skin regeneration. Please ensure that all crystals remaining on the skin are removed thoroughly before starting the treatment.

Step 3:

Apply disposable gloves. These serve your and the protection of your customers. Always wear gloves for the Microneedling. Then remove a fresh "MESO" pad from the packaging and screw it onto the "MESO" hand device.

How often should Microneedling applications be performed?

For an effective spa treatment, 6 to 12 treatments are necessary depending on the skin type. These should be done twice a week. The treatment should then be inserted into the regular treatment of the customer every 4 to 5 weeks in order to maintain the long-term effect.



Treatments & products?

Mesotherapie - one of several treatment courses:

Cleansing: All Jean D'Arce products except Dermal Oil Cleanser

Peeling: Various Jean D'Arce peeling, if necessary **Dermal AHA** neutralization

Deep peeling: **Exfoliating tonic + Dermal Tonic Spray**

Pre Needling: **Mono** concentrates according to skin type
without colorants, fragrance and preservatives

- * **Dermal Lift Repair Concentrate NEW**
- * **Dermal Hyaluron Booster Concentrate NEW**
- * **Dermal Hyaluron Filler Concentrate NEW**
- * **Dermal Mimic Lift Concentrate NEW**

Needling: **Dermal Hyaluron5 Gel [5-fold - NEW**

Post Needling: **Dermal Cooling Mask - NEW**
Dermal Power Peptide Mask

Conclusion: ARCELMED, Miratense , Renovar, Sensitive


Additionally: **Dermal Ultrasonic Contact Gel [with aloe vera NEW**



Medical Beauty Expertise

**Independent studies for treatment with MultiNeedling Pro
in a realistic treatment session;**


The results of our MultiNeedling Pro application were demonstrated in an independent study. We inform you about before-and-after results of our MultiNeedling Pro treatment, as well as the experience reports of our subjects.

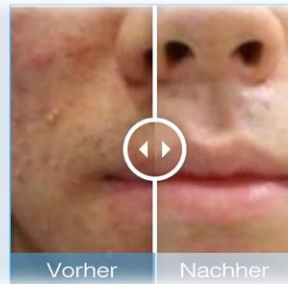



**Before-and-after result
in combination with Multineedling Pro
and Medical beauty products**

Medical Beauty Expertise

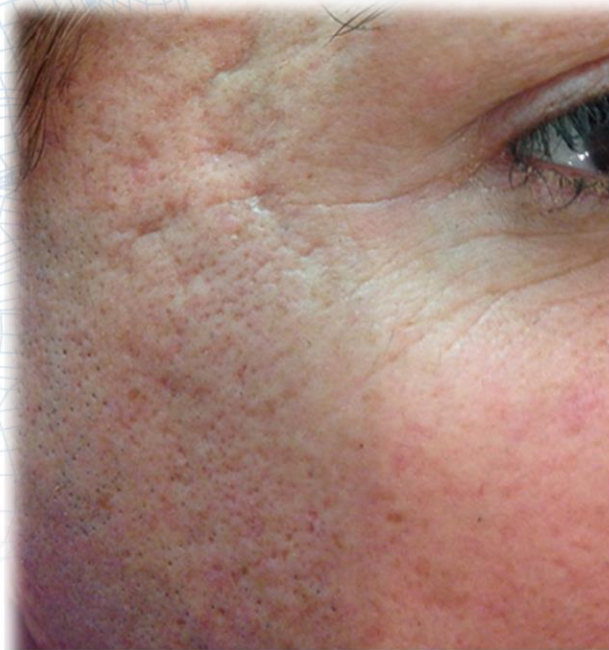

VOR DER BEHANDLUNG

Before treatment



NACH NUR ZWEI BEHANDLUNGEN

After two treatments




**Before-and-after result
in combination with Multineedling Pro
and Medical beauty products**

Medical Beauty Expertise



Before treatment

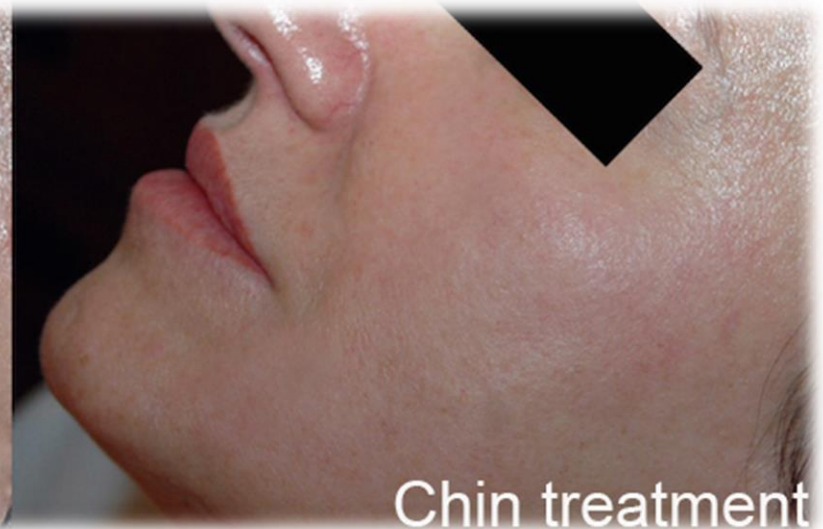

Chin treatment

After two treatments



**Before-and-after result
in combination with Multineedling Pro
and Medical beauty products**

Medical Beauty Expertise



Before treatment



acne-scarring treatment

After four treatments



**Before—and-after result
in combination with Multineedling Pro
and Medical beauty products**

Medical Beauty Expertise



Before treatment

After four treatments



Designed,
developed and
made in Germany


JEAN D'ARCEL
TECHNOLOGIE



Compared to other products,
Jean D'Arcel MultiNeedlig Pro is:

- More precise
- More gentle
- More effective

Medical Beauty



Multineedlig Pro
was presented to you by Christoph Walenda

BEAUTY IS OUR PROFESSION